

# HAAG-STREIT Contact Glasses

DISCOVER A DIFFERENT WORLD.

 **HAAG-STREIT**  
INTERNATIONAL

Precision by Tradition

## HAAG-STREIT Contact Glasses

For decades, only original Goldmann Contact Glasses have satisfied the Ophthalmologist's demands for quality all over the world.

They allow binocular observation of most parts of the human eye, even under the most unfavourable conditions.

**FOR ABSOLUTE CERTAINTY – DON'T GO FOR LESS THAN A HAAG-STREIT PRODUCT.**

Main features:


- ▶ unequalled quality workmanship providing superb optical surfaces
- ▶ total quality control system after each manufacturing step
- ▶ special bonding technique for an extra long life
- ▶ exceptional consistency in quality provided by our skilled specialists who have an average of more than ten years of experience in manufacturing our lenses



145 L

## Contact Glass RETINA 145 L

The Retina 145 L direct contact glass is a panfundus wide-angle viewing device designed to facilitate the problem-free diagnosis and treatment of the retina as far as the equator.

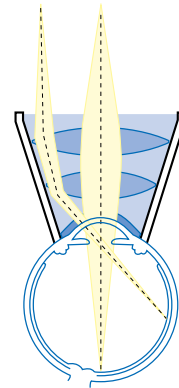



903



906



907



630

## ORIGINAL GOLDMANN THREE MIRROR GLASSES

MADE OF ACRYLIC GLASS

The best combination with the Haag-Streit 900® Slitlamps. For examination of the entire fundus, the vitreous and the iridocorneal angle.

### Contact Glass 903 and 903 S

The classic Goldmann Glass.

### Contact Glass 906 and 906 S

For newly born children.

10 mm internal diameter of the cornea part.


### Contact Glass 907

For children up to 4 years of age.

11 mm internal diameter of the cornea part.

### Contact Glass 630

For use with Slitlamps which have a smaller distance between the eye and the reduction prism than the Haag-Streit 900®.

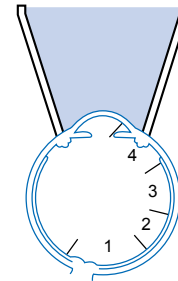

All Glasses have the original mirror angles by Goldmann: 59 / 66 / 73 degrees.

For disinfection refer to our instruction manual



**901**



**902**



**902 S**



**905**



**905 S**



**904**

## ORIGINAL GOLDMANN GLASSES


MADE OF ACRYLIC GLASS

### Fundus Glass

#### Contact Glass 901

For the binocular slit observation of the pupil, the macula, its surroundings out to 30° as well as the central vitreous body sections.


Easy to use, lightweight glass provides a direct image.

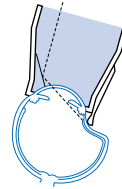

### Ora Serrata Glass

#### One mirror Contact Glass 904

This contact glass serves to examine the ora serrata, the pars plana and the extreme peripheral parts of the vitreous body and the ocular fundus.

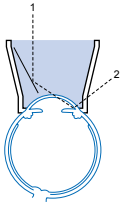

### Glaucoma Glasses

#### One mirror Contact Glass 902 and 902 S

For the examination of the anatomical conditions in the area of the front of the iris.

The expanse and form of the iridocorneal angle can only be ascertained by employing a narrow slit beam.

The slit lamp 900®, with its swivelling and declinable illumination, is primarily capable of providing an assessment of the expanse of the entire angle of the iris, which is of special importance in glaucoma diagnostics.

#### Two mirror Contact Glass 905 and 905 S

Two mirrored surfaces, which each incline at an angle of 62° towards the front surface, have been cut into the 17 mm long glass cone. This makes it possible to examine the opposite iridocorneal angle without turning the contact glass.

The expanse and form of the angle of the iris can be correctly ascertained by using a narrow slit on the slit lamp.

For disinfection refer to our instruction manual



**CGAL**



**CGIL**



**CGPL**



**CGRL**




**CGVL**




# Laser Contact Glasses

MADE OF MINERAL GLASS

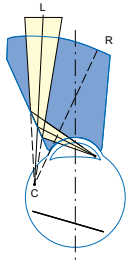


## Laser Contact Glass CGAL

The CGAL gonioscopic contact glass was designed by Roussel and Fankhauser for laser treatments of the chamber angle. It reduces the laser spot by a factor of 1.5 as compared to a Goldmann Glass.

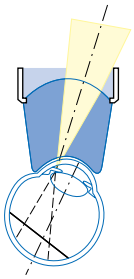


## Laser Contact Glass CGIL

The CGIL iridectomy contact glass was designed by Riquin, Fankhauser et al. to perform full thickness peripheral laser iridectomies.

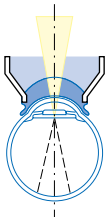


## Laser Contact Glass GPL

The GPL capsulotomy contact glass was designed by Riquin, Fankhauser et al. for the dissection of opacified posterior lens capsules and membranes in the pupillary and retropupillary space with the YAG laser.

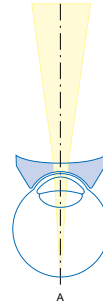

## Laser Contact Glass CGRL

The CGRL is a Wide Angle Contact Glass designed for laser photocoagulation and diagnosis of the retina.

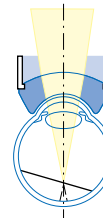

## Laser Contact Glass CGVL

The CGVL vitrectomy contact glass was designed by Rol, Fankhauser et al. for photodisruptive YAG laser procedures in the posterior vitreous.

For disinfection refer to our instruction manual



**903 L**



**906 L**



**907 L**



**630 L**

## ORIGINAL GOLDMANN THREE MIRROR GLASSES

MADE OF ACRYLIC GLASS WITH LUMENIZED-COATED  
PROTECTIVE GLASS

Designed to work with argon/YAG Lasers.

### Contact Glass 903 L

The classic Goldmann Glass.

### Contact Glass 906 L


For newly born children 10 mm internal diameter of the cornea part.

### Contact Glass 907 L

For children up to 4 years of age 11 mm internal diameter of the cornea part.

### Contact Glass 630 L

For use with Slitlamps which have a smaller distance between the eye and the reduction prism than the Haag-Streit 900®.

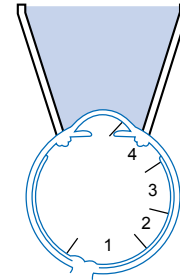

All Glasses have the original mirror angles by Goldmann: 59 / 66 / 73 degrees.

For disinfection refer to our instruction manual

**HAAG-STREIT AG**

Feinmechanik/Optik  
Gartenstadtstrasse 10  
CH-3098 Köniz  
Phone ++41 31 978 01 11  
Fax ++41 31 978 02 82  
info@haag-streit.ch  
www.haag-streit.com

**HAAG-STREIT  
DEUTSCHLAND GmbH**

Rosengarten 10  
D-22880 Wedel  
Phone ++49 4103 709 02  
Fax ++49 4103 709 370  
www.haag-streit.de

**HAAG-STREIT  
AUSTRIA GmbH**

Mariahilfer Strasse 182/5  
A-1150 Wien  
Phone ++43 1 895 0364  
Fax ++43 1 895 0368  
info@haag-streit.at  
www.haag-streit.at

**HAAG-STREIT  
USA INC**

5500 Courseview Drive  
Mason/Ohio 45040  
Phone ++1 513 336 7255  
Fax ++1 513 336 7260  
Toll Free 800 787 5426  
info@haag-streit-usa.com  
www.haag-streit.com

**HAAG-STREIT  
FRANCE**

P.A. de la Bouvarde  
Immeuble les Pléiades  
F-74370 Metz-Tessy  
Phone ++33 4 5009 0033  
Fax ++33 4 5009 7190  
info@haag-streit.fr  
www.haag-streit.fr



Precision by Tradition

Specifications subject to change without prior notice